


# Implant Kit Transition Guide:

## SPEEDSHIFT™ Implant Kit

BME Implant Kits available while supplies last

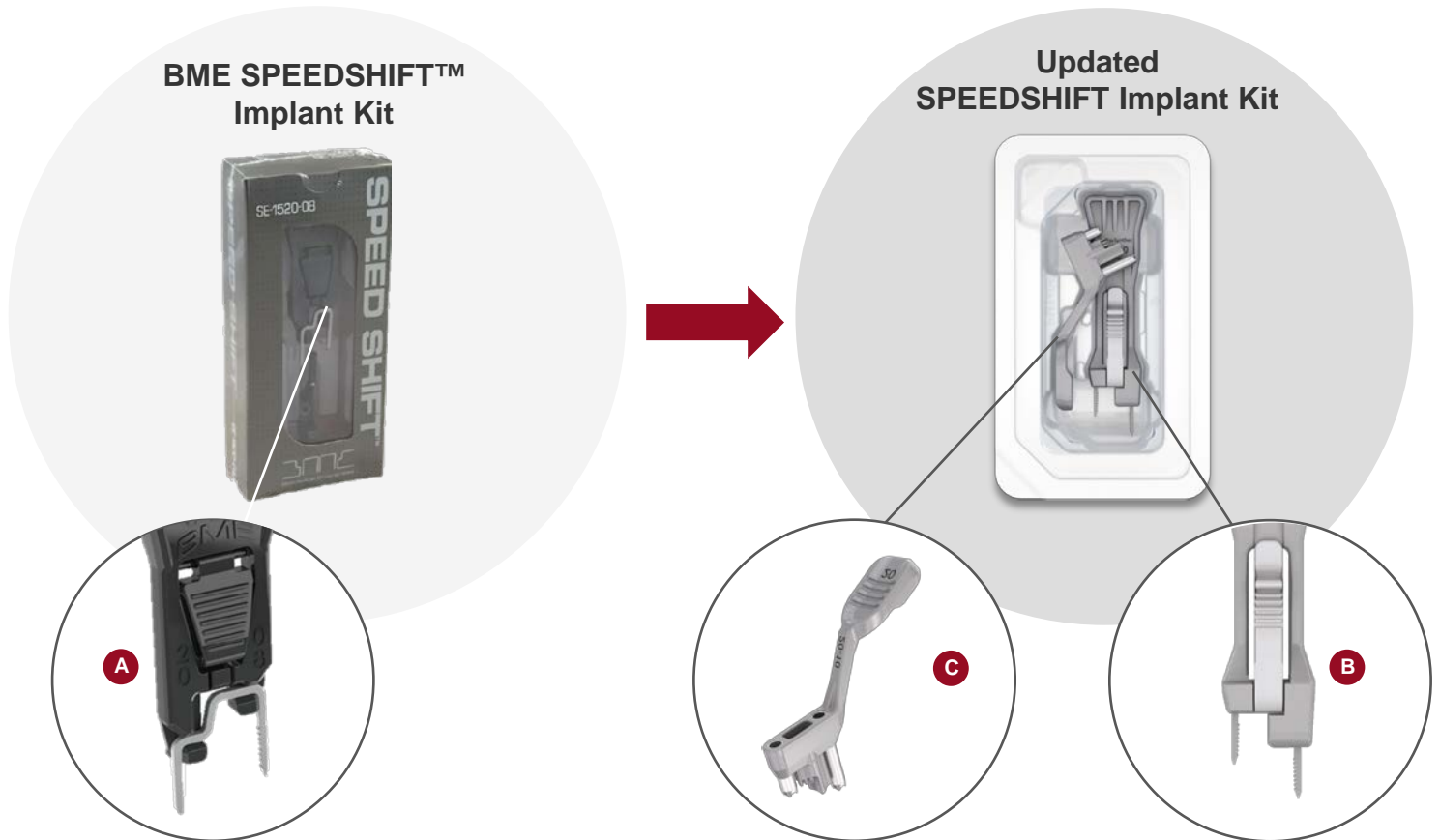

### What's Changed?

- Color Change: Black → Grey
- Insertion Tool Release Mechanism:
  - Push Button **A** → Slider **B**
  - See release mechanism instruction on next page
- One piece thumb-grip Drill Guide included **C**
  - Replaces Drill Guide Handle & Handle Adapter
  - Positive drill stop remains

# Implant Kit Transition Guide:

## SPEEDSHIFT™ Implant Kit

BME Implant Kits available while supplies last

### Existing BME SPEEDSHIFT™ Implant Kit



#### Push Button:

Press the distal third of the central button with the thumb while simultaneously twisting the Insertion Tool until disengagement has been verified

#### Additional Implant Disengagement Options:

- Press button with thumb to release
- Twist Insertion Tool in either direction to release

### Updated SPEEDSHIFT Implant Kit



#### Slider Release Instruction:

Pull and hold the slider button away from the Implant to release the Implant from the Insertion Tool.